

Caring for your wound

After DERMABOND™ PRINEO™ Skin Closure System has been applied

Your healthcare professional has chosen to use DERMABOND™ PRINEO™ System to close your wound¹

- DERMABOND™ PRINEO™ System is the combination of a mesh and a liquid adhesive that allows the incision or wound to be held together during the healing process.
- DERMABOND™ PRINEO™ System should remain in place until your healthcare professional has determined that adequate healing has occurred, which is usually anywhere between 7–14 days. In most cases, DERMABOND™ PRINEO™ System is easily removed with little or no discomfort.
- In the event that you notice that DERMABOND™ PRINEO™ System is beginning to loosen or may be coming off, contact your healthcare professional.

Things to know

Bathing or showering¹

If directed by your healthcare professional, you may occasionally and briefly wet your incision or wound that was treated with DERMABOND™ PRINEO™ System in the shower or bath. Do not soak or scrub your incision or wound. Do not swim or soak your incision or wound in water.

After showering or bathing, gently blot your incision or wound dry with a soft towel. If a dry protective dressing is being used over DERMABOND™ PRINEO™ System, it should be replaced with a fresh, dry protective dressing after showering or bathing as directed by your health care practitioner. Care should also be taken so that any tape that may be part of the dry protective dressing does not come into contact with DERMABOND™ PRINEO™ System because when the tape is removed, it may also remove DERMABOND™ PRINEO™ System.



Wound healing

If you experience any redness, swelling, discomfort, warmth or pus, contact your healthcare professional and he or she will determine how your incision or wound is healing and take the necessary steps to address any issues.

Exercise¹

Do not engage in strenuous physical activity that may cause additional stress on your incision or wound. Follow your healthcare professional's guidance about when you can return to your normal activities.

Removing DERMABOND™ PRINEO™ System¹

- Your healthcare professional will determine when the healing process of your incision or wound has been completed and DERMABOND™ PRINEO™ System is ready to be removed, which is usually between 7 to 14 days. When healing is complete, your healthcare professional will carefully peel off the DERMABOND™ PRINEO™ System.
- Prior to removal, do not scratch, rub or pick at the mesh. This may loosen the adhesive and mesh before the skin is healed.
- In the event that you notice that DERMABOND™ PRINEO™ System is beginning to loosen and may be coming off or comes off the skin/ wound, contact your healthcare professional.

Ointments or liquids¹

Topical ointments, liquids or any other product (other than dry bandages) should not be applied to the incision while DERMABOND™ PRINEO™ System is in place. These may loosen DERMABOND™ PRINEO™ System from the skin before it has completely healed.

This document is provided for general reference only and does not constitute medical advice. You are advised to discuss with your surgeon / general physician the applicability of this document.

Reference: 1. Ethicon Lab-0013100/Rev:6 DERMABOND™ PRINEO™ Skin Closure System, Instructions for Use. January 2020. Data on file.



Please always refer to the Instructions for Use / Package Insert that come with the device for the most current and complete instructions.

Johnson & Johnson MedTech

Johnson & Johnson AG
Gubelstrasse 34
6300 Zug
Schweiz
www.jnj.ch/medtech