

# Recommendations to Help Minimize Skin Reactions



Skin reactions (i.e., redness, itchy rashes, blisters, etc.) may occur after the use of DERMABOND™ PRINEO™ System. Using the recommendations below, which are based on the Instructions for use, the potential for these reactions may be minimized.

## Before Applying the Product – Patient Selection

**DERMABOND™ PRINEO™ System should not be used in patients with a known hypersensitivity to cyanoacrylate, formaldehyde, benzalkonium chloride (BAC), or pressure-sensitive adhesives.<sup>1</sup>**


Because these ingredients can also be found in non-medical devices such as household or beauty products, ask your patient about any previous skin reactions to the following products as this may indicate a sensitivity to the ingredients in DERMABOND™ PRINEO™ System.

<b>Cyanoacrylate</b>	Super glue, False eyelashes, False ingernails			
<b>Formaldehyde</b>	Paints, Varnishes, Industrial adhesives (e.g., carpet glue)			
<b>Benzalkonium chloride (BAC)</b>	Cosmetics, Cleaning wipes Antibacterial adhesive bandages			
<b>Pressure-sensitive adhesives</b>	Adhesive bandages, Athletic tape, First-aid tape			


# Product Application

Properly apply the topical skin adhesive over the mesh.


1. Using the adhesive applicator, **spread the liquid adhesive smoothly and evenly over the entire length of the mesh**, covering only a small margin of surrounding skin.<sup>1</sup>

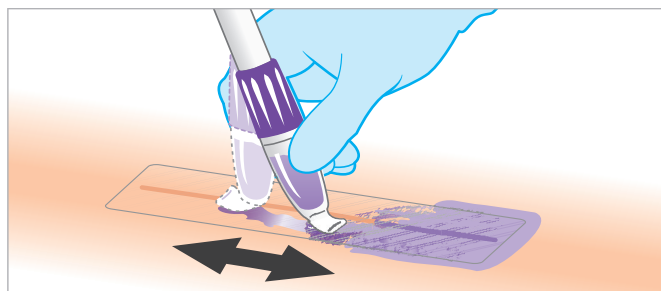

## CORRECT

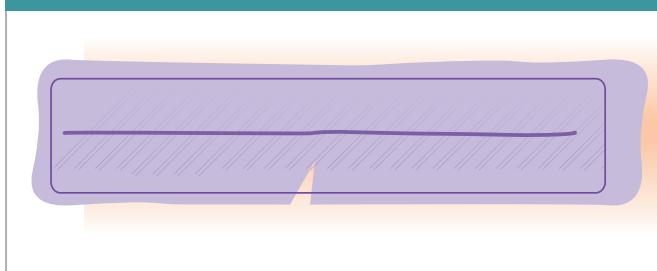


## INCORRECT

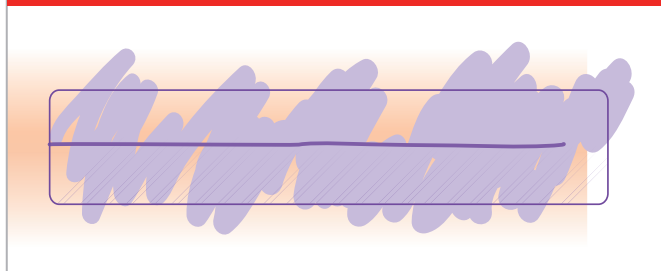


2. Using sterile gauze, quickly **wipe away any excess liquid adhesive that has run o the mesh**.

## CORRECT

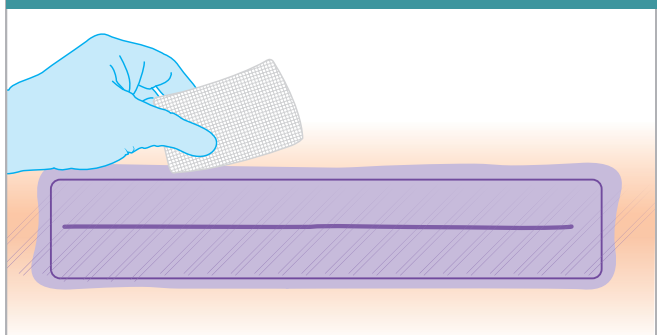

## INCORRECT

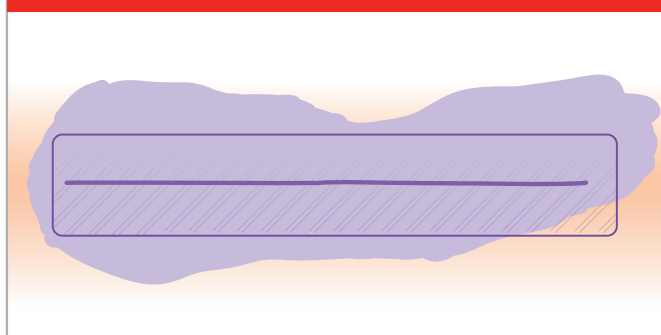

**Reference 1:** ETHICON 100926837 DERMABOND™ PRINEO™ Skin Closure System 22cm. Instructions for Use. 15 November 2023. ADAPTIV doc: 380457. Data on File

**Please refer always to the Instructions for Use / Package Insert that come with the device for the most current and complete instructions.**

The third-party trademarks used herein are trademarks of their respective owners.

Johnson & Johnson AG  
Gubelstrasse 34  
6300 Zug  
Schweiz  
[www.jnj.ch/medtech](http://www.jnj.ch/medtech)

© Johnson & Johnson AG 2025. C\_CH\_SRG\_WOUN\_137994.1.EN